

OWL Mirror Set-Up Instructions



The OWL mirror will arrive fully assembled in a separate box. See reverse for re-packing and return instructions.

STEP 1

When you remove the mirror from the box, it should look similar to this.



STEP 2

Position the mirror so the small mirror is away from you and the large mirror is close to you. Peer over the top of the large mirror down into the small mirror. Here, the OWL Mirror is shown being used with the Seated Support.



STEP 3

Slide the mirror back towards the chair so that the small U shaped notched out area goes back to the arm rest strap. Adjust as needed.



NOTE:

It is important that you follow your doctor's instructions to look straight down as much as possible. This product is designed to allow you to see objects in a normal manner while your eyes are looking straight down. The OWL Mirror is designed to be very adjustable and easy to use. You may clean the mirror with any window cleaner or soft towel.

STEP 4

Once your head is properly positioned with your eyes looking straight down, you will be able to see what is directly across from you. By pivoting the base and top of the mirror and moving the mirror back and forth on the work desk, you may adjust the mirror as needed to insure a good view.



Here you can see the OWL Mirror being used with a Table Top Face Support.

NOTE: Generally the mirror must be located as close to your body as possible.



Mirror Instructions continued

How to re-pack for shipment

OWL MIRROR RE-PACKING INSTRUCTIONS:

Patients are responsible for the safe return of equipment. All items must be packaged properly for the return trip. If there are any questions as to how to repack, follow the steps below or call your Personal Care Consultant for assistance. If it arrives at Oakworks damaged, the patient is responsible for replacement costs.

STEP 1

Position the mirror as shown.



STEP 2

Wrap bubble wrap around the outside of the frame meeting on the inside between the two mirrors. (Note - mirror surface is covered.)



STEP 3

Place mirror flat in box.



STEP 4

Fold the top of the box "over under" as shown. Place box inside larger equipment box in any remaining available space after larger equipment has been packed.



923 East Wellspring Road
New Freedom, PA 17349
Toll Free: 800.916.4592
www.owleasing.com